

Your quick guide to using NovoPen® 6

Insulin dose
recording
happening here



NovoPen® 6



This is a quick guide. Please see the Instructions for Use that come with NovoPen® 6 for full information.

Prepare your NovoPen® 6



Insert an insulin cartridge

Pull off the pen cap. Twist off the cartridge holder. The piston rod may stick out of your pen. If so, push it in until it stops.



Slide a new insulin cartridge into the cartridge holder with the threaded end first. Twist the cartridge holder back on.



Attach a new needle


Tear off the paper tab. Push the needle straight onto the pen. Turn until it is on tight. Pull off both needle caps. Save the outer needle cap.

How to use a NovoPen® 6

Watch this quick guide video to learn how to prepare for and complete your injections with NovoPen® 6.



Give your injection

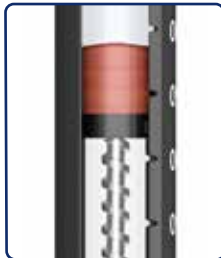


⚠ ALWAYS test the insulin flow

Select 2 units and press the dose button until the dose counter shows "0". The insulin flow test is complete when you see insulin squirt and there is no longer a gap between the piston rod head and the piston.



Select your dose

Pull out the dose button. Make sure the dose counter shows "0" before you start. Turn the dose button to select the dose you need.

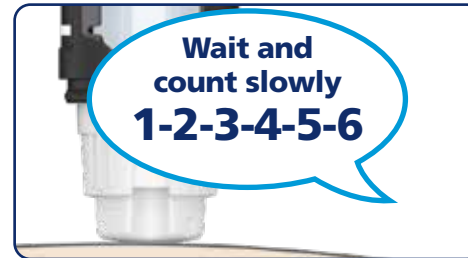

Inject your dose

Insert the needle. Press the dose button until the dose counter shows "0". Count slowly to 6 before you remove the needle from your skin. Safely remove and throw away the needle. Put the pen cap back on.



Scan the QR code to watch the video

How to check your last dose



Pull out the dose button

Make sure the dose counter shows "0" before you start.



Push the dose button back in

The dose memory will now show the details of your last dose.



Read your dose memory

The example above shows 12 units injected 1 hour, 52 minutes and 3 seconds ago.



NovoPen® 6 is a registered trademark of Novo Nordisk A/S.
Novo Nordisk Pharmaceuticals Pty Ltd. ABN 40 002 879 996. Level 10, 118 Mount Street, North Sydney, NSW, 2060. Novo Nordisk Customer Care (Australia) 1800 668 626. www.novonordisk.com.au
© Registered trademark of Novo Nordisk A/S. AU23NP600002. NONOCH0478. Date of Preparation: April 2023.

NovoPen® 6 offers a 60-unit maximum dose in 1-unit dose increments.

NovoPen® 6)